



H series urine analyzer

Calibration liquid for Turbidity

Instructions

【Product name】 H series urine analyzer Calibration liquid for Turbidity

【Dimension】 8ml/vial

【Intended Use】 H series urine analyzer Calibration liquid for Turbidity is used for H Series Urine Analyzer, FUS Series Urinalysis Hybrid and MUS Series Urinalysis System's turbidimeter calibration.

【Principle】 Using scattering theory, turbidimeter tests calibration liquid.

【Composition】 Fu MA hydrazine turbidity quality control liquid (800NTU) .

【Storage Conditions & Shelf Life】 Stored at 2°C~8°C in a dry place and protected from sunlight. Refer to the label for the shelf life. After opened, stored at 2°C~8°C, the shelf life is 30 days.

【Applicable Instrument】 H Series Urine Analyzer, FUS Series Urinalysis Hybrid and MUS Series Urinalysis System.

【Procedure】 According to H Series Urine Analyzer, FUS Series Urinalysis Hybrid and MUS Series Urinalysis System user manuals.

【Result explanation】 After turbidimeter calibration, the screen will show "calibration Ok" or "calibration not OK" . "Calibration OK" means the instrument is in normal status; "Calibration not OK" means the calibration failed, please rerun calibration or clean the turbidimeter.

【Warning and precautions】 Please blend well and avoid contact with skins. In case of skin contact, rinse with water. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. If swallowed, induce vomiting and seek medical advice.

【Product performance】 accuracy: Turbidity=400±30NTU

【Date of Instruction Approval and Modification】 04/2023

【Explanation for Symbols】



Please read package insert



Store at



Manufactured by



Batch code



In Vitro Diagnostic Use



Authorised Representative



Catalogue Number



The calibration liquid for Turbidity conform to the directive 98/79/EC (IVD-directive)



Expiry date

DIRUI INDUSTRIAL CO.,LTD.

95 Yunhe Street, New & High Tech.

Development Zone Changchun, Jilin 130012 P.R.China

Tel: +86 431 85100409

Fax: +86 431 85172581

E-mail: dirui@dirui.com.cn

http://www.dirui.com.cn



IVD



Emergo Europe B.V. Westervoortsedijk 60
6827 AT Arnhem The Netherlands