



Cleaning liquid II

(Concentrated type)

Instruction

【Product Name】 Cleaning liquid II

【Packing Specification】 200mL, 500mL, 1L

【Intended Use】 For the cleaning and rinsing of the tubing system.

【Test Principle】 Flow cell technique and high-speed imaging technique

【Main Ingredient】 Lowfoaming surfactant

【Storage Condition and Shelf life】

Stored at 2°C~30°C in a dry place, protected from sunlight, and sealed. For shelf life refer to the label.
The open-vial shelf life is 30 days.

【For Instrument】 FUS Series Urinalysis Hybrid and MUS Series Urinalysis System.

【Usage】 Take 1 part of detergent and dilute with the purified water with 19 parts. The cleaning method should refer to the user manual of Automatic Urine Analysis System.

【Sample Requirement】 Fresh urine sample without centrifugation

【Performance Indicator】 After being diluted PH is 7.60 ± 1.00 at $(25 \pm 1^\circ\text{C})$

【Matters Needing Attention】

1. Avoid contact with skin and eyes, in case of contact with skin, rinse the contact area with water immediately, and see a doctor in a necessary;
2. Tighten the cap after use;
3. Samples, waste liquid, etc. potentially bioinfectious. The operators should follow the laboratory safety operation regulations and treat the waste liquid according to the regulations of local medical waste, infectious waste and industrial waste.

【Bibliography】

1. Ferris JA Comparison and standards of urine microscopy Lab Med 14:659-662, 1983
2. Method Comparison and Bias Estimation Using Patient Samples; Approved Guideline-Second Edition.

【Symbol Explanation】



Please read instruction



Storage Temperature



Keep away from sunlight



Catalogue Number



Expiry



Lot No



Authorised Representative



In Vitor Diagnosis



Manufacturer



European In Vitro Diagnostic Medical Device Directive 98/79/EC(IVDD)

【Manual Approved and Modified Date】 12/2018



DIRUI INDUSTRIAL CO.,LTD.

95 Yunhe Street, New & High Tech.

Development Zone Changchun, Jilin 130012 P.R.China

Tel: +86 431 85100409

Fax: +86 431 85172581

E-mail: dirui@dirui.com.cn

http://www.dirui.com.cn



EMERGO EUROPE Prinsessegracht 20
2514 AP The Hague The Netherlands